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4 ENGLISH

Dear customer

Thank you for choosing HESS Medizintechnik AG. You will enjoy this long-lasting product. We attach great importance to our focus on quality, functionality and ergonomics. We guarantee you a careful selection of materials, the use of modern production technology and constant quality control. The products of Hess Medizintechnik AG meet safety and hygiene standards and fulfil the high demands of everyday hospital and practice use at all times.

Follow the instructions for use. This will allow you to take full advantage of our product and at the same time prevent improper use.

Your HESS Medizintechnik AG

4.1 Symbols and definitions

4.1.1 Notes on the instructions for use

Correct operation of the appliance is essential for safe operation. Therefore, please familiarise yourself with the contents of these operating instructions before using the appliance. Persons who are not familiar with the operating instructions or have not been instructed in their use are not permitted to use the appliance.

 The operating instructions must be accessible to the operating personnel at all times in order to avoid operating errors and ensure trouble-free operation.

 HESS Medizintechnik AG offers technician training courses for maintenance and servicing work on the products. A certificate obtained as part of this training subsequently authorises participants to carry out technical work on our products independently.

4.1.1.1 Pictograms used

The pictograms used are described below:

Attention!
General warning. Pay particular attention to these passages to avoid injury and damage.

Important note!
A safety instruction serves to protect people and other living beings or the environment from hazards.

Information!

Follow the operating instructions!
Read the instructions or the relevant section of the instructions carefully before operating or manipulating the appliance.

Disconnect the mains plug from the power supply to avoid electric shock!

4.1.2 Regulations and laws

This appliance has been developed and manufactured in accordance with applicable regulations. This ensures a very high level of safety.

The model described here is a Class 1 medical device and complies with the following guidelines:


- Regulation (EU) 2017/745 on medical devices (MDR)
- Safety with regard to irritation/allergies according to EN ISO 10993-10 tested materials

4.2 Safety instructions

- Read and follow the instructions for use before using the product
- Observe the safety instructions in the respective chapter
- This appliance may only be used for its intended purpose (see chapter 4.4.1)
- The product may only be used by persons with instruction or under supervision. Do not allow children to play with it.
- Do not use the product on sloping surfaces.
- Check the mains cable carefully for damage before connecting the product to the mains
- Position the product so that the appliance plug can be easily removed from the socket!
- When operating electrical and mechanical functions, ensure that no persons or objects are in the adjustment range or are endangered by them
- The maximum load capacity must not be exceeded. Can be seen in the technical data (see section 4.4.2)
- As the user, make sure that the appliance is in perfect condition before each start-up
- In the event of irregularities or defects, the device must be taken out of operation until the defect has been rectified and the customer service of HESS Medizintechnik AG must be contacted (see chapter 4.4.5)
- Always use original spare parts from HESS Medizintechnik AG
- No manipulations or modifications may be made to the device or parts thereof, otherwise the guarantee will be invalidated
- All serious incidents occurring in connection with the product must be reported immediately to the manufacturer and the competent authority!

4.2.1 Type plate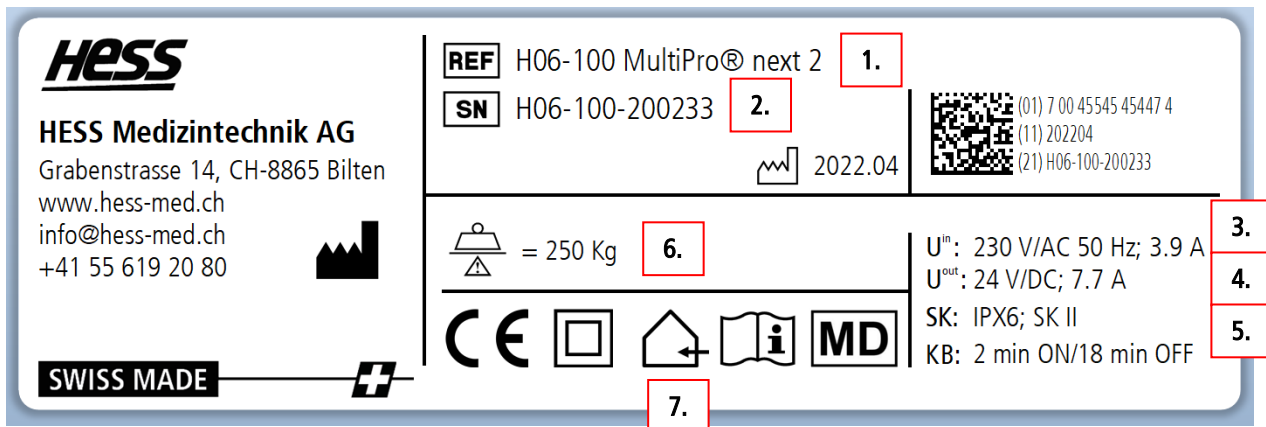

Example of a type plate for the MultiPro examination, treatment and therapy table® in the standard version with mains connection. Please note the type plate on your table.

1. Model designation
2. Serial number
3. Mains voltage; Mains frequency; Power consumption
4. Protection of electrical components against splash water from all sides
5. Operating time of the electric motorised adjustment; to protect against overheating, please observe this information! In the example, the motorised height adjustment may be moved for a maximum of 2 minutes during 20 minutes.
6. Load capacity
7. Explanation of the symbols used on the rating plate:

	Conformity mark according to Medical Device Regulation 2017/745
	Protection class II (double insulation, protective insulation)
	"Only use in dry rooms"
	Read the instructions carefully before operating or tampering with the appliance.
	Manufacturer of the medical device in accordance with the EN ISO 15223-1:2021 directive
	Medical Device: Is a medical device

4.3 Cleaning, disinfection and maintenance

The following instructions for cleaning and disinfection are intended as a guide. Only use cleaning agents and disinfectants at temperatures between 5°C and 50°C.

Important note!

Always observe the manufacturer's instructions for the cleaning agent or disinfectant used!

4.3.1 Cleaning

The surfaces are extremely easy to clean. A mild soap solution is sufficient to clean the upholstery and frame. For stubborn stains, we recommend our cleaning agent HESS MultiClean (Art. H06-991). Please refer to the description on the bottle label.

Cleaning process:

1. Use a cloth with a lukewarm soap solution and wipe off any dirt with the cloth.
2. Use a separate, clean cloth with clear water and wipe off any soap residue.
3. If necessary, dry the upholstery with a cloth.

4.3.1.1 Cleaning the upholstery cover (if available)

The upholstery cover has a so-called **sanitised surface with a permanent anti-microbial, anti-bacterial and anti-mycotic effect**.

Important note

- Correct and regular cleaning extends the life of the upholstery cover.
- The service life of the upholstery cover may be impaired if disinfectant is used incorrectly. Please refer to the chapter on disinfectants.
- Do not place any colouring, hot or sharp objects on the upholstery. This can damage the upholstery cover.
- Denim fabrics in particular will permanently discolour the upholstery cover.

The guarantee only covers production faults. Discolouration and embrittlement of the upholstery cover are excluded.

4.3.1.2 Cleaning the frame

Never use abrasive cleaners, scouring pads or stainless steel care products to clean the frame. For lacquered surfaces, we recommend HESS MultiClean, according to the instructions on the label. After each cleaning, we recommend wiping the frame with a damp cloth and drying it.

4.3.2 Disinfection

We recommend the disinfectants according to IFU00_Cleaning and disinfection of HESS covers. Pay particular attention to the solution concentration and proper handling in accordance with the manufacturer's instructions. Disinfectants can affect the service life of the upholstery cover if used incorrectly. Before using other disinfectants, ask the manufacturer of the disinfectant about the suitability of the product.

Attention!

It is essential to ensure correct handling. Failure to do so can lead to brittleness, discolouration and cracking of the upholstery cover within a short time!

4.3.3 Safety inspection STK

Our products are low-maintenance. The user is obliged to carry out a safety inspection (STK in accordance with EN 62353) once a year. If defects are found during the safety inspection that restrict the safe operation of the product, the product must be taken out of service until all defects have been rectified.

The STK is carried out in accordance with the document STK-Protocol-Universal. This can be found in the appendix to these instructions for use.

The manufacturer is only liable for the safety and reliability of the product if it is regularly maintained and used in accordance with its intended purpose.

Attention!

Always disconnect the product from the power supply before carrying out maintenance and repair work. Maintenance work that goes beyond the scope described may only be carried out by trained technicians!

Our service department will be happy to help you: **e-mail: service@hess-med.ch** or **service number +41 55 619 20 87**

4.4 Use of the product

4.4.1 Intended use

The BaseLine next examination, therapy and treatment tables from HESS Medizintechnik AG are height-adjustable patient tables and are used for the temporary positioning of patients for examination, therapy and treatment purposes. The positioning of the patient can be optimally adjusted using adjustment mechanisms. In principle, the tables can be used in all indoor areas, but ideally in medical or therapeutic environments such as clinics, surgeries and other similar institutions. The table is designed exclusively for this purpose. The safety instructions (chapter 2) must be observed to ensure safe use.

4.4.2 Technical data

Product*	BaseLine examination, therapy & treatment table - Class I medical device
Basic UDI-DI	764046632BaseLine8E
Mains voltage**	100/120/230V AC 50-60 Hz 3.9A
Output voltage**	24V DC / 8A - Protection class 2
Control box**	Protection class: IPX4 tested to EN 60601-1/UL 60601-1
Motor**	Protection class: IPX4 tested in accordance with EN 60601-1 and UL 2601-1
Short-term operation (KB)**	2/20: max. operating time 2 min. then 18 min. break
Max. load capacity:	150 kg
Weight:	approx. 75-85 kg

*Product-specific information on the type plate / **Not valid for the pump models

4.4.3 Delivery

HESS-Medizintechnik AG products are delivered fully assembled and ready for use. If this is not the case, our service department can help you.

4.4.4 Commissioning

Before putting the product into operation, check it for damage and defects. Check that the table is stable.

Observe the following points when designing your product.

- **Products without electrical components** can be used immediately.
- **Products with electrical components** must be connected to a suitable power source before use.
- **Battery-operated products** should be fully charged before use.

If the product has a lock box, you must ensure that it is unlocked.

Important note!

Always use a suitable power source with an RCD.

4.4.5 Decommissioning

To switch off the appliance, remove the mains cable from the socket and activate the lock box if your product has one.

4.4.5.1 Professional disposal

Old appliances are not worthless waste! Valuable raw materials can be recovered through environmentally friendly disposal.

Important note!

Batteries and rechargeable batteries must be disposed of separately.

To preserve and protect the environment, prevent pollution and improve the reuse of raw materials (recycling), dispose of this product after use in accordance with the laws in force in your country.

Attention!

Failure to observe the applicable regulations has an impact on the environment!

There is a risk of fire if the battery is not disposed of correctly!

4.4.6 Ambient conditions


The following environmental conditions must be met during use of the product.

Storage temperature	-10°C - 50°C
Operating temperature	5°C - 40°C
Air humidity	20% - 90%, at 30° (not condensed)
Pressure	700 hPa - 1060 hPa

4.5 Operation

4.5.1 Function overview

Before using this product, the user must ensure that it is safe to use and in proper condition. Check that the table is stable.

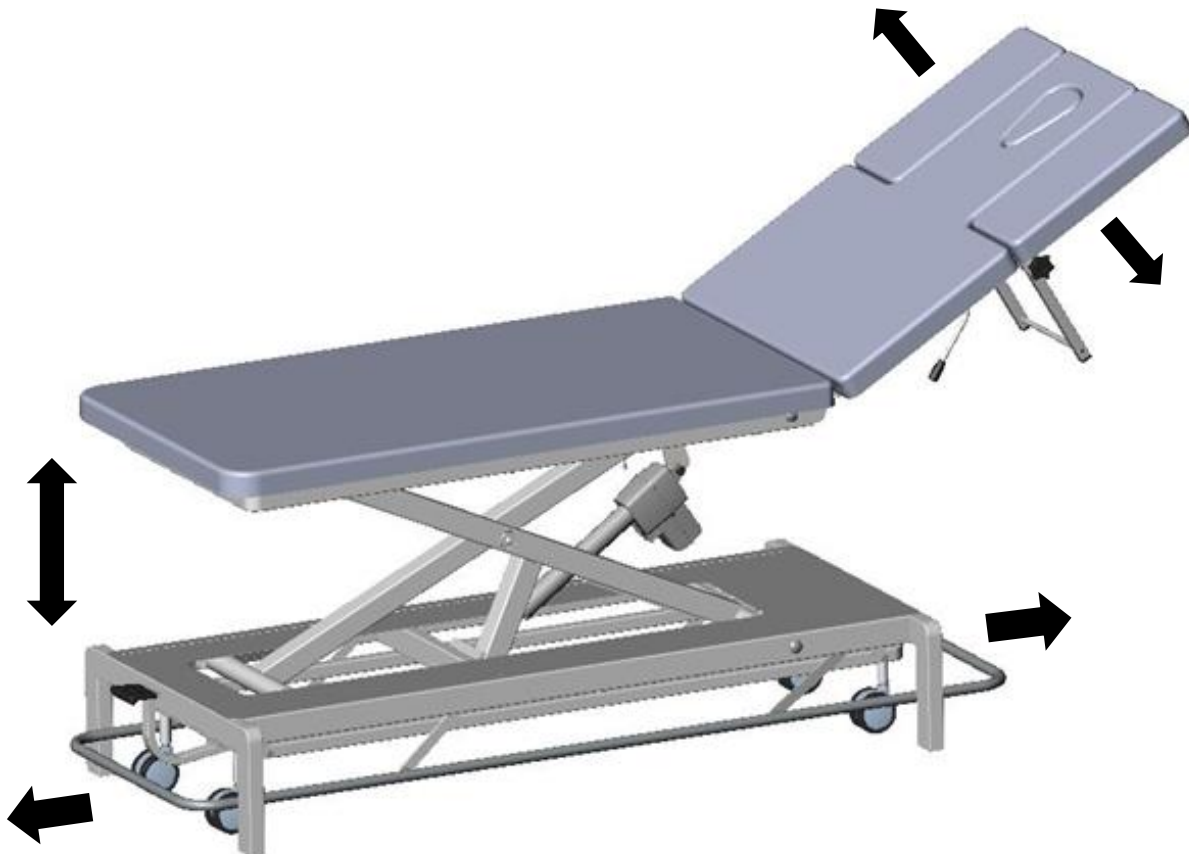
4.5.2 Travelling with the table


Always remove the mains cable before moving the table!

Press the pedal (1) down with your foot until it engages. The table can now be moved freely. Press the pedal (1) again to set the table back on its feet and lock it.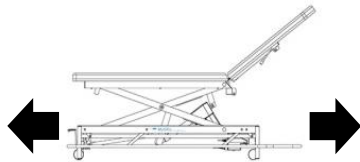

Bewegen der Liege
déplacer le divan d'examen
spostare il divano dell' esame
moving the examination table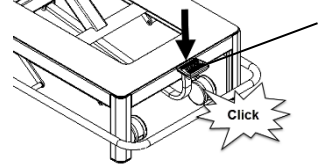

Important note!


The table must not be moved with the patient.
Do not use the table on sloping surfaces.

4.5.3 Height adjustment

Caution!

Before adjusting the height of the bed, make sure that it is positioned on the feet and is stable. Otherwise the patient could injure themselves.

Press the up arrow on the handset to move the bed upwards or the down arrow to move the bed downwards.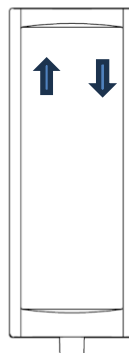

Caution!


- Make sure that there are no people or objects in the adjustment range of the recliner.
- The electrical functions are not suitable for continuous operation for more than 2 minutes. The built-in thermal protection switches the drive off automatically to prevent overheating. If this is the case, you must wait approx. 60 minutes until further operation is possible again. (Note the rating plate!)

4.5.4 Upholstery adjustment General

All gas spring-supported adjustments can be set using a lever.

To set the desired position, move the lever (1) by pushing or pulling against the cushion (2).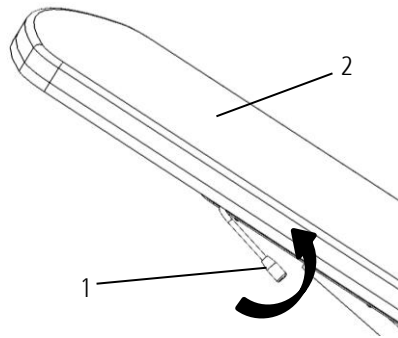


Important note!

The back section may be loaded with a maximum of 100kg.

4.5.5 Arm adjustment (BaseLine next 5)

Turn the star grip (1) anti-clockwise to release the armrest and adjust the position of the armrest by pulling the armrest (2) up or down.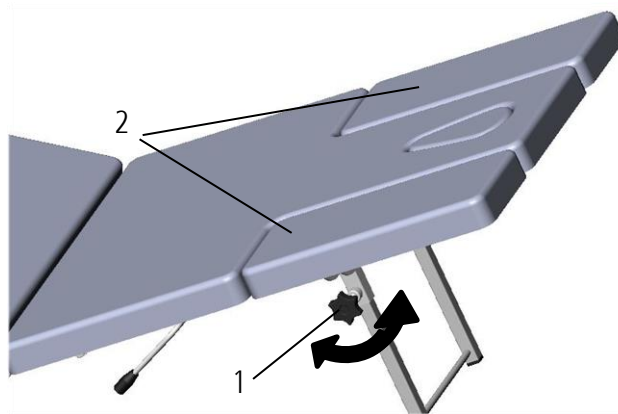

Important note!

The armrests may be loaded with a maximum of 40kg each.


4.5.6 All-round switching bracket for height adjustment (option H04-502)

Important note!

Before adjusting the height of the bed, make sure that it is positioned on the feet and is stable. Otherwise the patient could injure themselves.

Push or pull the all-round handle (1) down or up to move the lounger up and down.

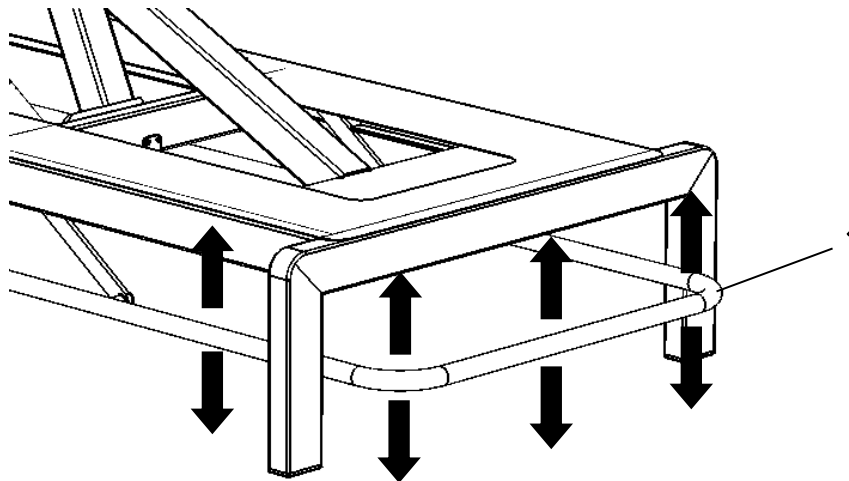Caution!

- Ensure that there are no persons or objects in the adjustment range of the table.
 - The electrical functions are not suitable for continuous operation for more than 2 minutes. The built-in thermal protection switches the drive off automatically to prevent overheating. If this is the case, you must wait approx. 60 minutes until further operation is possible again. (Note the rating plate!)
-

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When it comes to furnishing and equipping hospitals, surgeries and nursing homes, we are your competent partner. From standard to specific customer solutions, you get everything from a single source.

Consulting / Development / Sales / Service

HESS Medizintechnik AG

Grabenstrasse 14
CH-8865 Bilten
T +41 55 619 20 80
info@hess-med.ch
hess-med.ch

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Copyright®